




PRIMARY PACKAGING/ INDIVIDUALLY SEALED CONTAINER

Brand Name or Identity of Manufacturer/Importer

LOT and/or : (LOT number format as per defined in K-WI-QA-02)

EXP and/or  : YYYY-MM or YYYY/MM

CE mark (*minimum height of 5mm with notified body's number*)

Any additional marking required by local regulations



SECONDARY PACKAGING/ CONSUMER BOX

Remark: Equivalent statement may be used to convey the information in this secondary packaging; however, it shall carry the same meaning

FRONT PANEL

Brand Name [logo]

[Sub-brand/ Types]

No. of condoms contain in the package

BACK/ LEFT SIDE/ RIGHT SIDE PANELS

Carex® logo
[Types]

[With reservoir tip]
[Nominal width]
[Shape and Texture]
[Lubricated or Dry]
[Coloured or Natural]
[Flavours] *(Required if flavoured)*
[Nominal Length] *(Required if less than 160mm)*
[Drawing of a condom] *(Optional to convey all above information)*

Carex® [Types/ Texture] condoms are designed for pleasure and reliability.

- Conform to ISO 4074

When used correctly every time you have sex, condom intended for contraception and prevention of sexually transmitted infections.

This product contains natural rubber latex which may cause allergic reactions including anaphylactic shock if the user is allergic to latex. OR



Condom is medical device OR



Use each condom only once. OR



For maximum benefit, carefully follow the condom instructions inside this package. OR



Store in a cool, dry place away from direct sunlight. OR



DO NOT USE OIL-BASED LUBRICANTS, such as petroleum jelly, vegetable/mineral oils or cold cream, as these may damage the latex condom.

100% ELECTRONICALLY TESTED

QUALITY ASSURANCE
[logo]



Manufacturer

OR



Karex Industries Sdn. Bhd.
PTD 7906 & 7907, Taman Pontian Jaya,
Batu 34, Jalan Johor, 82000 Pontian,
Johor Darul Takzim, Malaysia.

CE mark

(minimum height of 5mm with notified body's number)

EU Authorized
Representative

OR



Advena Limited
Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta

Importer

OR




: **XXX company & address**




UDI or other format [Barcode]

(When AIDC carriers other than the UDI carrier are part of the product labelling, the UDI carrier shall be readily identifiable and UDI symbol may be used as appropriate. Otherwise, the use of UDI symbol shall not be obligatory)

BOTTOM PANEL

LOT and/or  : (LOT number format as per defined in K-WI-QA-02)

EXP and/or  : YYYY-MM or YYYY/MM



INSTRUCTION FOR USE (IFU)

Remark: Information or statements in this IFU may be rephrased accordingly; however, it shall carry the same meaning.

FRONT

IMPORTANT POINTS TO KNOW BEFORE USING A CONDOM

1. If extra lubrication is needed, use water-based lubricants.
2. **DO NOT USE OIL BASED LUBRICANTS** such as petroleum jelly, baby oil, body lotions, massage oils, butter, margarine, etc., as they may weaken the condom and cause subsequent failure of the product. If you're using topical medicines, please consult a doctor or pharmacist about the compatibility of topical medicines that might come in contact with the condom.
3. Do not use the condom if the rubber materials are sticky or brittle. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.
4. Seek medical assistance as soon as possible, at least within 72 hours should a condom leak or burst during use.
5. Condoms are for single use only; reusing condom could increase the risk of breakage and infection.
6. When used correctly every time you have sex, latex condoms help prevent pregnancy, reduce the risk of transmitting HIV/AIDS and other sexually transmitted infection.
7. No method of contraception can provide 100% protection against pregnancy or transmission of HIV and STIs. If you suspect that you or your partner may have become pregnant or infected, consult your doctor immediately.
8. This product contains natural rubber latex which may cause allergic reactions including anaphylactic shock. If you are allergic to natural rubber latex products, consult your doctor for advice before use.
9. Store in a cool dry place away from direct sunlight.
10. Keep away from children.
11. There is a risk of choking using a condom orally.
12. The use of additional lubricants during anal sex may reduce the risk of breakage.
13. Condom is a medical device intended to be used by lay persons - adult male or female.
14. Report any serious incident occurred from using the device to the manufacturer and the relevant national competent authority.
15. These condoms conform to International Standard ISO 4074/ EN ISO 4074.



Karex Industries Sdn. Bhd.
PTD 7906 & 7907, Taman Pontian Jaya,
Batu 34, Jalan Johor, 82000 Pontian,
Johor Darul Takzim, Malaysia.

Carex® [logo]

Instruction For Use

CE mark

(minimum height of 5 mm with notified body's number and correct font, to be placed beside manufacturer's detail)

Revision Date: 10th October 2024 (v.1)

Master reference information: K-IFU-01/1 (v.1)




BACK


Instruction For Use




1. Carefully tear off foil pack, making sure not to damage the condom.

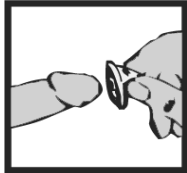


2. During removal from foil, avoid damage to the condom by fingernails, jewelry and etc.

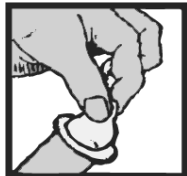

3. Squeeze the reservoir end of the condom gently, between the thumb and the forefinger to expel the air out of the tip, while placing the condom over the head of the erect penis. This must be done before any contact occurs between the penis and the partner's body to assist in the prevention of sexually transmitted infectious and pregnancy.

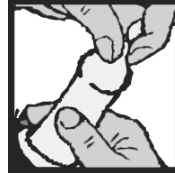

4. While still holding the reservoir tip, unroll the condom over the entire length of the erect penis with the other hand.

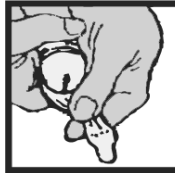

5. Stop and check if the condom slipping off or tightening excessively onto the penis as this might lead to breakage.



6. After ejaculation, hold on the rim of the condom and slowly withdraw the penis while it is still erect so that the condom does not slip off.

7. Remove the condom from your penis by sliding it off slowly. It is important not to permit the penis or used condom to touch the vagina area after withdrawal.

8. Wrap the used condom and dispose it hygienically in a closed rubbish bin.

	Store in a cool, dry place		Keep away from sunlight		Use by date
	Contains natural rubber latex		Manufacturer		Consult instructions for use
	Importer		Do not reuse		Batch code
	European authorized representative		Medical device		Unique Device Identifier

Remark: Add in definition of any other symbols used on the packaging




Document History


Version No	Author	Date	Description of Change
1	Nurul Syazwani	10.10.2024	New format for draft label K-IFU-01/1 (normal intended use)



PRIMARY PACKAGING/ INDIVIDUALLY SEALED CONTAINER

Brand Name or Identity of Manufacturer/ Importer

LOT and/or  : (LOT number format as per defined in I-WI-QA-02)

EXP and/or  : YYYY-MM or YYYY/MM

CE mark (minimum height of 5mm with notified body's number)

Any additional marking required by local regulations.



SECONDARY PACKAGING/ CONSUMER BOX

Remark: Equivalent statement may be used to convey the information in this secondary packaging; however, it shall carry the same meaning

FRONT PANEL

Brand Name [logo]

[Sub-brand/ Types]

No. of condoms contain in the package.

BACK / LEFT SIDE/ RIGHT SIDE PANELS

Brand Name logo
[Types]

[With reservoir tip]
[Nominal width]
[Shape and Texture]
[Lubricated or Dry]
[Coloured or Natural]
[Flavours] *(Required if flavoured)*
[Nominal Length] *(Required if less than 160mm)*
[Drawing of a condom] *(Optional to convey all above information)*

Brand Name [Types/ Texture] condoms are designed for pleasure and reliability.

- Conform to EN ISO 4074.

When used correctly every time you have sex, condom intended for contraception and prevention of sexually transmitted infections.

This product contains natural rubber latex which may cause allergic reactions including anaphylactic shock if the user is allergic to latex. OR



Condom is medical device OR



Use each condom only once. OR



For maximum benefit, carefully follow the condom instructions inside this package. OR



Store in a cool, dry place away from direct sunlight. OR




DO NOT USE OIL-BASED LUBRICANTS, such as petroleum jelly, vegetable/mineral oils or cold cream, as these may damage the latex condom.

100% ELECTRONICALLY TESTED

QUALITY ASSURANCE
[logo]



Manufacturer
OR


Innolates Sdn. Bhd.
Lot 591 & 594, Persiaran Raja Lumu,
Pandamaran Industrial Estate,
42000 Port Klang, Selangor, Malaysia.

CE mark
(minimum height of 5mm with notified body's number)

EU Authorized
Representative
OR

Advena Limited
Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta

EC REP


Importer
OR


: **XXX company & address**

UDI

UDI or other format [Barcode]
(When AIDC carriers other than the UDI carrier are part of the product labelling, the UDI carrier shall be readily identifiable and UDI symbol may be used as appropriate. Otherwise, the use of UDI symbol shall not be obligatory)

BOTTOM PANEL

LOT and/or  : (LOT number format as per defined in I-WI-QA-02)


EXP and/or  : YYYY-MM or YYYY/MM



INSTRUCTION FOR USE

Remark: Information or statements in this IFU may be rephrased accordingly; however, it shall carry the same meaning.


FRONT

<p>IMPORTANT POINTS TO KNOW BEFORE USING A CONDOM</p> <ol style="list-style-type: none">1. If extra lubrication is needed, use water-based lubricants.2. DO NOT USE OIL BASED LUBRICANTS such as petroleum jelly, baby oil, body lotions, massage oils, butter, margarine, etc., as they may weaken the condom and cause subsequent failure of the product. If you're using topical medicines, please consult a doctor or pharmacist about the compatibility of topical medicines that might come in contact with the condom.3. Do not use the condom if the rubber materials are sticky or brittle. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.4. Seek medical assistance as soon as possible, at least within 72 hours should a condom leak or burst during use.5. Condoms are for single use only; reusing condom could increase the risk of breakage and infection.6. When used correctly every time you have sex, latex condoms help prevent pregnancy, reduce the risk of transmitting HIV/AIDS and other sexually transmitted infection.7. No method of contraception can provide 100% protection against pregnancy or transmission of HIV and STIs. If you suspect that you or your partner may have become pregnant or infected, consult your doctor immediately.8. This product contains natural rubber latex which may cause allergic reactions including anaphylactic shock. If you are allergic to natural rubber latex products, consult your doctor for advice before use.9. Store in a cool dry place away from direct sunlight.10. Keep away from children.11. There is a risk of choking using a condom orally.12. The use of additional lubricants during anal sex may reduce the risk of breakage.13. Condom is a medical device intended to be used by lay persons - adult male or female.14. Report any serious incident occurred from using the device to the manufacturer and the relevant national competent authority.15. These condoms conform to International Standard ISO 4074/ EN ISO 4074. <p> Innolatex Sdn. Bhd. Lot 591 & 594, Persiaran Raja Lumu, Pandamaran Industrial Estate, 42000 Port Klang, Selangor, Malaysia.</p>	<p>Brand Name [logo]</p> <p>Instruction For Use</p> <p>CE mark <i>(minimum height of 5 mm with notified body's number and correct font, to be placed beside manufacturer's detail)</i></p> <p>Revision Date: 10th October 2024 (v.6) Master reference information: I-IFU-01 (v.6)</p>
---	---










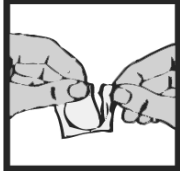



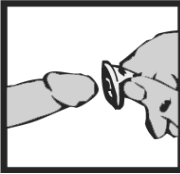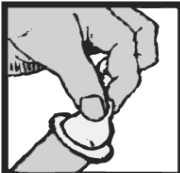

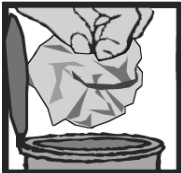



BACK PANEL

Instruction For Use



- Carefully tear off foil pack, making sure not to damage the condom.
- During removal from foil, avoid damage to the condom by fingernails, jewelry and etc.
- Squeeze the reservoir end of the condom gently, between the thumb and the forefinger to expel the air out of the tip, while placing the condom over the head of the erect penis. This must be done before any contact occurs between the penis and the partner's body to assist in the prevention of sexually transmitted infectious and pregnancy.
- While still holding the reservoir tip, unroll the condom over the entire length of the erect penis with the other hand.
- Stop and check if the condom slipping off or tightening excessively onto the penis as this might lead to breakage.
- After ejaculation, hold on the rim of the condom and slowly withdraw the penis while it is still erect so that the condom does not slip off.
- Remove the condom from your penis by sliding it off slowly. It is important not to permit the penis or used condom to touch the vagina area after withdrawal.
- Wrap the used condom and dispose it hygienically in a closed rubbish bin.

	Store in a cool, dry place		Keep away from sunlight		Use by date
	Contains natural rubber latex		Manufacturer		Consult instructions for use
	Importer		Do not reuse		Batch code
	European authorized representative		Medical device		Unique Device Identifier

Remark: Add in definition of any other symbols used on the packaging




Document History


Version No	Author	Date	Description of Change
05	Nurul Syazwani	10.07.2024	Revised instruction for use.
06	Nurul Syazwani	10.10.2024	Rearrange content and format of draft label. Remove 'Do not use if Package is damaged' symbol from section IFU – Back side.



PRIMARY PACKAGING/ INDIVIDUALLY SEALED CONTAINER

Brand Name or Identity of Manufacturer/Importer

LOT and/or  : (LOT number format as per defined in M-WI-QA-02)

EXP and/or  : YYYY-MM or YYYY/MM

CE mark (*minimum height of 5mm with notified body's number*)

Any additional marking required by local regulations



SECONDARY PACKAGING/ CONSUMER BOX

Remark: Equivalent statement may be used to convey the information in this secondary packaging; however, it shall carry the same meaning

FRONT PANEL

Brand Name [logo]

[Sub-brand/ Types]

No. of condoms contain in the package

BACK / LEFT SIDE/ RIGHT SIDE PANELS

Brand Name logo
[Types]

[With reservoir tip]
[Nominal width]
[Shape and Texture]
[Lubricated or Dry]
[Coloured or Natural]
[Flavours] *(Required if flavoured)*
[Nominal Length] *(Required if less than 160mm)*
[Drawing of a condom] *(Optional to convey all above information)*

Brand Name [Types/ Texture] condoms are designed for pleasure and reliability.

- Conform to EN ISO 4074.

When used correctly every time you have sex, condom intended for contraception and prevention of sexually transmitted infections.

This product contains natural rubber latex which may cause allergic reactions including anaphylactic shock if the user is allergic to latex. OR



Condom is medical device OR



Use each condom only once. OR



For maximum benefit, carefully follow the condom instructions inside this package. OR



Store in a cool, dry place away from direct sunlight. OR



DO NOT USE OIL-BASED LUBRICANTS, such as petroleum jelly, vegetable/mineral oils or cold cream, as these may damage the latex condom.


100% ELECTRONICALLY TESTED

QUALITY ASSURANCE
[logo]



MEDICAL-LATEX (DUA) SDN. BHD.

M-IFU-01
Rev 10th October 2024
(v.4)

Manufacturer
OR


Medical-Latex (DUA) Sdn. Bhd.
Plo 8, Senai Industrial Estate,
81400 Senai,
Johor, Malaysia.

CE mark
*(minimum height of 5mm with notified body's
number)*

EU Authorized
Representative
OR

EC	REP
----	-----

Advena Limited
Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta

Importer
OR


: **XXX company & address**




UDI or other format [Barcode]
(When AIDC carriers other than the UDI carrier are part of the product labelling, the UDI carrier shall be readily identifiable and UDI symbol may be used as appropriate. Otherwise, the use of UDI symbol shall not be obligatory)

BOTTOM PANEL

LOT and/or

LOT

 : (LOT number format as per defined in M-WI-QA-02)

EXP and/or  : YYYY-MM or YYYY/MM



INSTRUCTION FOR USE

Remark: Information or statements in this IFU may be rephrased accordingly; however, it shall carry the same meaning.

FRONT

Important Information/Safety Precautions:

- A condom is for a single use only. If re-use there may be an increased risk of failure or infection.
- Store it in a cool, dry place away from sunlight. Heat can deteriorate the physical properties of condoms.
- **DO NOT USE OIL BASED LUBRICANTS** such as petroleum jelly, mineral oil and cold cream, as they may weaken the condom and cause subsequent failure of the product. Always use water based or latex compatible lubricants.
- Consult a doctor or a pharmacist if you or your partner is using prescription or over-the-counter medication that is applied to the penis or vagina and may come into contact with the condom.
- In case of latex allergy, condoms should only be used after consulting a doctor. The condoms are made of natural rubber latex, which might cause allergic reactions including anaphylactic shock.
- When used correctly every time you have sex, latex condoms help prevent pregnancy, reduce the risk of transmitting HIV/AIDS and other sexually transmitted infectious.
- No contraception can guarantee 100% protection against pregnancy and sexually transmitted diseases.
- Non-vaginal use of condoms increases the risk of the condoms slippage or being damaged. Seek medical assistance, at least within 72 hours should a condom leak or burst.
- Store in a cool dry place away from direct sunlight.
- Keep away from children.
- The use of additional lubricants during anal sex can reduce the risk of breakage. There is a risk of choking using condom orally.
- Condom is a medical device to be used by lay persons - adult male or female.
- Report any serious incident occurred on the condom to the manufacturer and the relevant national competent authority.
- These condoms conform to International Standard ISO 4074/ EN ISO 4074.



Medical-Latex (DUA) Sdn. Bhd.
Plo 8, Senai Industrial Estate,
81400 Senai,
Johor, Malaysia.

Brand Name® [logo]

CE mark

(minimum height of 5 mm with notified body's number and correct font, to be placed beside manufacturer's detail)

Revision Date : 10th October 2024 (v.4)

Master reference information : M-IFU-01 (v.4)



BACK













How To Use A Condom

Prior to sex:

- Use a new condom each time you have sex. Always check the expiry date on the pocket.
- Take your condom out of the packet carefully, making sure that you don't damage it with your fingernails or jewellery.
- Always put the condom on as soon as the penis is erect but before there is any contact with the woman's genital area. This is important because fluid, which may contain sperm, can seep from the penis early during sex.
- Gently squeeze the tip, (the closed end) between your finger and thumb, to expel any trapped air and make space for the man's semen. Unroll the condom over the full length of the penis.
- Check if condom is slipping off/tightening excessively onto the penis as this may lead to breakage.
- Do not use the condom if the rubber material is sticky or brittle or the individual container is obviously damaged. Use a new one from an undamaged package.

After sex:

- After the man has ejaculated but before his erection is completely lost, hold the condom rim firmly around the penis while the penis is withdrawn. This makes sure that the condom doesn't slip, accidentally spilling semen in around the woman's vagina.
- To dispose the used condom, tie a knot in the end of the used condom to seal it. Wrap it in a tissue and dispose it hygienically. Do not flushed it down the toilet.

	Consult Instructions for Use		Batch code
	Contains natural rubber latex		Manufacturer
	Do not reuse		European Authorized representative
	Store in a cool, dry place		Use By / Expiry Date
	Keep away from sunlight		Importer
	Medical device		Unique Device Identifier

Remark: Add in definition of any other symbols used on the packaging.




Document History


Version No	Author	Date	Description of Change
04	Nurul Syazwani	10.10.2024	Rearrange content and format of draft label. Remove 'Do not use if Package is damaged' symbol and 'Trash Bin' symbol from section IFU – Back side. Amend definition of 'Latex' and 'Do Not Reuse', EU AR symbol



PRIMARY PACKAGING/INDIVIDUALLY SEALED CONTAINER

Brand Name or Identity of Manufacturer/Importer

“LOT” and/or  : *(LOT number format as per defined in T-WI-QA-02)*

“EXP” and/or  : YYYY-MM or YYYY/MM

CE mark *(minimum height of 5 mm with notified body's number)*

Any additional marking required by local regulations.



SECONDARY PACKAGING/CONSUMER BOX

Remark: Equivalent statement may be used to convey the information in this secondary packaging; however, it shall carry the same meaning

FRONT PANEL

**Brand Name [logo]
[Sub-brand/ Types]
No. of condoms contain in the package**

BACK/LEFT SIDE/ RIGHT SIDE PANELS

Brand Name ® logo [Types]	[Nominal width] [Colours or Natural] [Shape and Texture] [Lubricated or Dry] [Nominal Length] <i>(Required if less than 160mm)</i> [Flavours] <i>(Required if flavoured)</i> [Drawing of a condom] <i>(Optional to convey all above information)</i>
------------------------------	--

Brand Name ® [Type /Texture] condoms are designed for pleasure and reliability

- Conform to EN ISO 4074

When used correctly every time you have sex, condom intended for contraception and prevention of sexually transmitted infections.

This product contains natural rubber latex which may cause allergic reactions including anaphylactic shock if the user is allergic to latex.

OR



Condom is medical device

OR



Use each condom only once

OR



For maximum benefit, carefully follow the condom instructions inside this package.

OR



Store in a cool, dry place away from direct sunlight.

OR



DO NOT USE OIL-BASED LUBRICANTS, such as petroleum jelly, vegetable/mineral oils or cold cream, as these may damage the latex condom.

100% ELECTRONICALLY TESTED

QUALITY
ASSURANCE [logo]



“Manufacturer”



Innolutex (Thailand) Limited
E1-6, Export Processing Zone,
Southern Industrial Estate,
Village 4, Chalung Sub-district,
Hat Yai District,
Songkhla, 90110,
Thailand

CE mark

*(minimum height of 5mm with notified
body's number)*

“EU Authorized
Representative”

OR



Advena Limited
Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta.

“Importer”

OR



XXX company & address



UDI or other format [Barcode]

(When AIDC carriers other than the UDI carrier are part of the product labelling, the UDI carrier shall be readily identifiable and UDI symbol may be used as appropriate. Otherwise, the use of UDI symbol shall not be obligatory)

BOTTOM PANEL

“LOT” and/or : (LOT number format as per defined in T-WI-QA-02)


“EXP” and/or : YYYY-MM or YYYY/MM



INSTRUCTION FOR USE (IFU)

Remark: Information or statements in this IFU may be rephrased accordingly; however, it shall carry the same meaning.







FRONT

<div style="text-align: center;">IMPORTANT POINTS TO KNOW BEFORE USING A CONDOM</div> <ol style="list-style-type: none"> 1. If extra lubrication is needed, use water-based lubricants. 2. DO NOT USE OIL BASED LUBRICANTS such as petroleum jelly, mineral oil and cold cream, as they may weaken the condom and cause subsequent failure of the product. So can some topical medicines that are applied to the penis or vagina. If you are not sure about the effect of the medicines you’re using on condoms, ask your doctor or pharmacist. 3. Do not use the condom if the rubber materials are sticky or brittle. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package. 4. Seek medical assistance as soon as possible, at least within 72hour should a condom leak or burst during use. 5. Condoms are for single use only; reused condom could increase the risk of breakage and infection. 6. When used correctly every time you have sex, latex condoms help prevent pregnancy, reduce the risk of transmitting HIV/AIDS and other sexually transmitted infectious. 7. No method of contraception can provide 100% protection against pregnancy or transmission of HIV and STIs. If you suspect that you or your partner may have become pregnant or infected, consult your doctor immediately. 8. This product contains natural rubber latex which may cause allergic reactions including anaphylactic shock. If you are allergic to natural rubber latex products, consult your doctor for advice before use. 9. Store in a cool dry place away from direct sunlight. 10. Keep away from children. 11. There is a risk of choking using a condom orally. 12. The use of additional lubricants during anal sex can reduce the risk of breakage. 13. Condom is a medical device intended to be used by lay persons - adult male or female. 14. Report any serious incident occurred from using the device to the manufacturer and the relevant national competent authority. 15. These condoms conform to International standards ISO 4074/ EN ISO 4074 <div style="margin-top: 20px;">  <p>Innolatex (Thailand) Limited E1-6, Export Processing Zone, Southern Industrial Estate, Village 4, Chalung Sub-district, Hat Yai District, Songkhla, 90110, Thailand</p> </div>	<p>Brand Name ® [logo]</p> <p>Instruction for Use</p> <p>CE mark <i>(minimum height of 5 mm with notified body’s number and correct font, to be placed beside manufacturer’s detail)</i></p> <p>Revision Date: 11th October 2024 Master reference information: T-IFU-01 (V.8)</p>
---	--





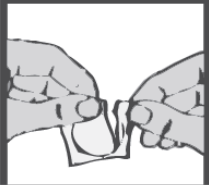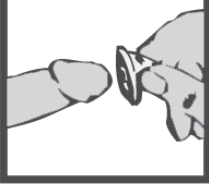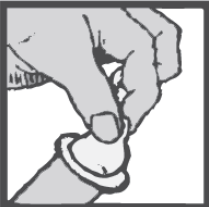




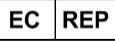




BACK

Instruction For Use

- Carefully tear off foil pack, making sure not to damage the condom.
- During removal from foil, avoid damage to the condom by fingernails, jewelry and etc.
- Squeeze the reservoir end of the condom gently, between the thumb and the forefinger to expel the air out of the tip, while placing the condom over the head of the erect penis. This must be done before any contact occurs between the penis and the partner's body to assist in the prevention of sexually transmitted infections and pregnancy.
- While still holding the reservoir tip, Unroll the condom over the entire length of the erect penis with the other hand.
- Stop and check if the condom slipping off or tightening excessively onto the penis as this might lead to breakage.
- After ejaculation, hold on the rim of the condom and slowly withdraw the penis while it is still erect so that the condom does not slip off.
- Remove the condom from your penis by sliding it off slowly. It is important not to permit the penis or used condom to touch the vagina area after withdrawal.
- Wrap the used condom and dispose it hygienically in a closed rubbish bin.

	Store in a cool, dry place		Keep away from sunlight		Use by Date
	Contains natural rubber latex		Manufacturer		Consult Instruction for Use
	Importer		Do not reuse		Batch code
	European authorized representative		Medical Device		Unique Device Identifier

Remark: Add in definition of any other symbols used on the packaging



Document History

Version No	Author	Date	Description of Change
06	Apinya Jiwrapat	24.08.2023	Revised instruction for use.
07	Apinya Jiwrapat	30.07.2024	Rearrange format.
08	Apinya Jiwrapat	11.10.2024	<ul style="list-style-type: none">- Rephrase statement of the latex symbol- Remove the EC REP symbol and detail on the Instruction for Use (Front)- Remove the symbol and definition of do not use if package is damaged